

Quick Start Guide



For more information visit www.listentech.com

Setup Instructions

Remove the clear plastic pull tab and fully charge the battery. When fully charged the LED is solid, indicating 100% charged.

Connect the Earphone/ Neck Loop Lanyard and ear speaker or headphones to the output jacks.

Charging:

Place the unit in the charging tray/case or connect the Micro USB to any standard USB port.

We Care

Hearing Protection:

This product is designed to amplify audio to a high volume level which could potentially cause hearing damage if used improperly. To protect your hearing, make sure the volume is turned down before putting on the ear speaker or headphones, then adjust the volume to your preference. Children should be supervised when using this product.

Congratulations

This Quick Start Guide will explain the basics of using your receiver. Experience how Listen Technologies brings power and clarity to the sounds that enrich our lives.

Button Basics

Power:

To turn on the receiver press and hold the power button for 1 second, to turn off the receiver, press and hold for 3 seconds.

Channel Select:

Press the programmable channel select button to cycle through active channels, seek an active channel, or lock a channel.

To change the default channel press and hold both of the volume buttons at the same time for 5 seconds.

Display:

Displays the battery status, unit ID, channel status, and charging status.

Medical Device Safety

Before using this product with an implantable or other device consult with your physician or manufacturer of the device. Always make sure you are using this product in accordance with the safety guidelines established by your physician or the manufacturer.

Recycling

Please help us protect the environment by properly disposing of the batteries. Take batteries or equipment to any electronic recycling collection point or return them to the factory.



© 2016 Listen Technologies Corporation®
All Rights Reserved 20160906

For further details regarding use, adjustment, or programming of your Listen products visit us at www.listentech.com/support/manuals or contact Listen at **+1.801.233.8992** or **1.800.330.0891**

To purchase additional accessories visit:
www.listentech.com/shop-listen